

Compass | Prophylaxis

Recommendations for the effective use of prophylaxis instruments in the dental practice



SonicLine

Sonic tips

1 Use | **Scaling** (up to 2 mm)



2 Use | **Scaling** (up to 4 mm)



PiezoLine

Ultrasonic tips **EM1** –
compatible with EMS units



Ultrasonic tips **SI1** –
compatible with Sirona® units



Ultrasonic tips **KA1** –
compatible with KaVo PIEZOlux™ units






Ultrasonic tips **ST1** –
compatible with Satelec® and NSK units




Polishers and brushes

3 Use | **Polishing**

Polishers (latex free)

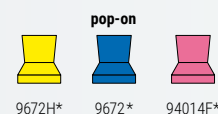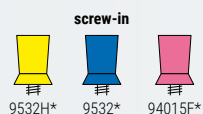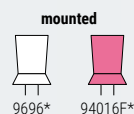

Brushes and mandrels

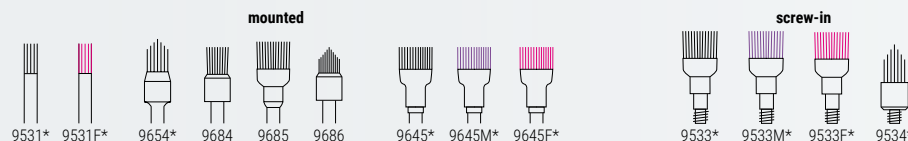

Polishers and brushes

Appropriate use

- Always work with low contact pressure to minimize the generation of heat.
- Always polish with gentle, circular movements.

Speed

☞_{opt.} 1.500 rpm

Excessive speed leads to an inferior surface quality.

The **maximum permissible speed** is indicated on the packaging and in the catalogue. **Failure to observe the maximum permissible speed leads to an increased safety risk!**

Sonic and ultrasonic tips

Appropriate use

- Make sure to use the correct power setting. For information on the right setting, please refer to the enclosed instructions.
- Start up ultrasonic tip prior to application to the tooth, supplying an adequate amount of spray cooling. All scalers require plenty of cooling.
- The tips should be used with low contact pressure. Excessive contact pressure reduces the oscillations of the tip and consequently its efficiency.

